

Parcel Folding :


The most common wrapping techniques that are applied for packaging of textile packs and instrument sets are the parcel fold. The unfolded wrapper covers the instrument table and thus provides a sterile field.

Used for bigger packages such as instrument trays, textile packs etc.




1.

Place item in center of wrapper.

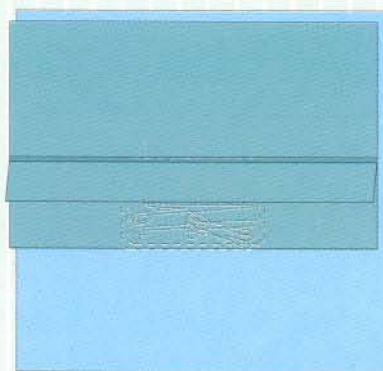


2.

Fold section nearest you to center and fold back cuff.

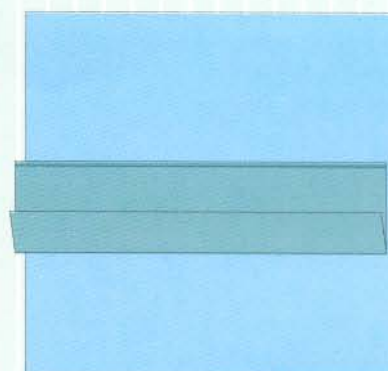


3.

Fold farthest section of wrapper to center and fold back cuff.

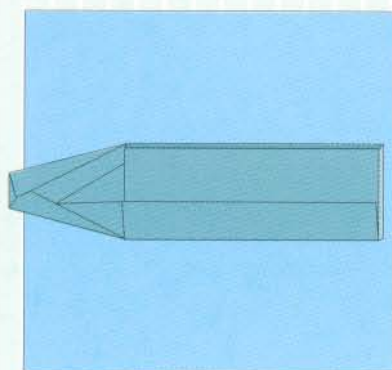


4.

Fold left section of wrapper to center and fold.

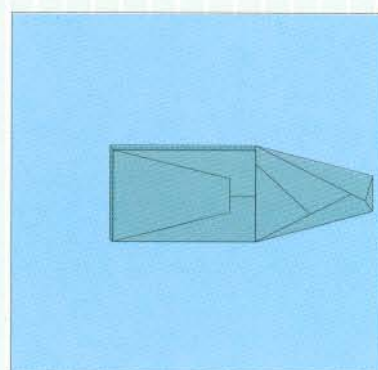

5.

Fold right section of wrapper to center and fold.

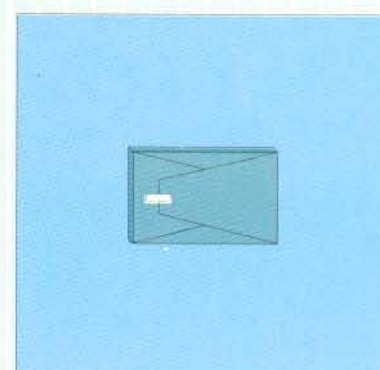

6.

Secure closure with adhesive tape.

For further information feel free to contact us :

SURGEINE HEALTHCARE (INDIA) PVT. LTD.

101, Vardhman Tru Plaza, CU Block, Pitampura, New Delhi-110088 (India)

Tel. : +91-11-47552462, 27348301 Telefax : +91-11-47552424

Mob. : +91-9350990935, 91-9310990931

E-mail : sales@surgeine.com


Website : www.surgeine.com

- CE 1023
- ISO 11607
- EN 868-2
- ISO 13485
- GMP
- ISO 9001


When opening a pack containing sterile materials, it is essential that, due to the act of opening, the content is not contaminated. Wrapping techniques for packs and sets have been developed which assures this aseptic opening.

Envelope Folding :


Envelope folding is the most common and recommended because it creates more tortuous path (more double bends) which means a better barrier against penetration of micro-organism. It also allows very simple aseptic opening of the pack with reduced handling thanks to the tab. The pack is most often closed with an adhesive tape which might have on it indicators for sterilization.

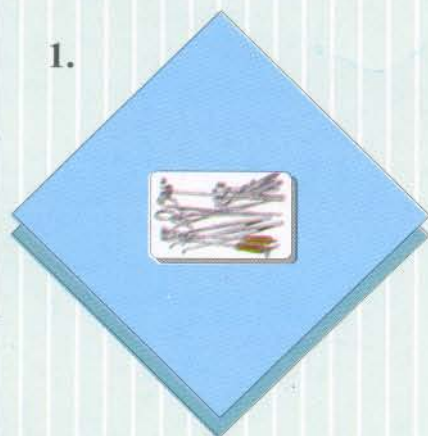


Place item in center of wrapper

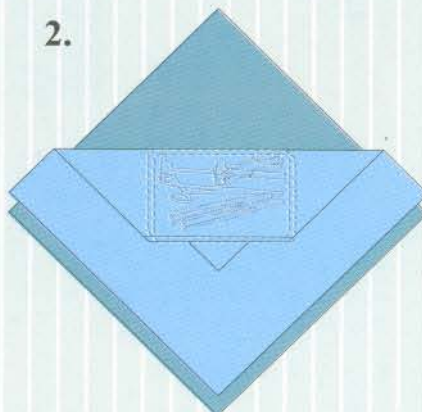


Fold section nearest you to center and fold back point

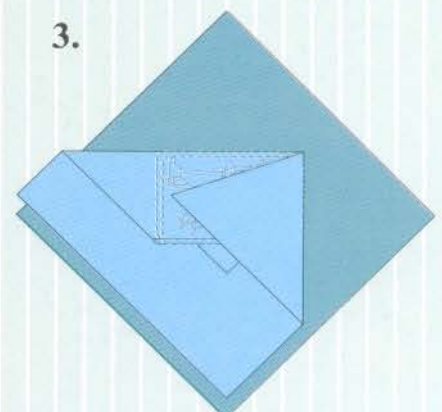


Fold left section to center and fold back point

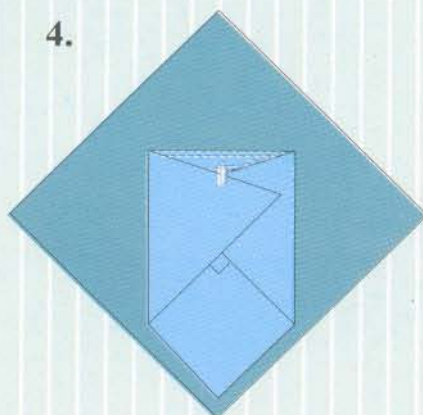

Fold right section to center and fold back point

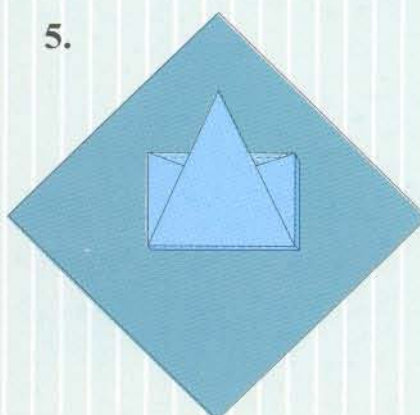

Fold top section to center.

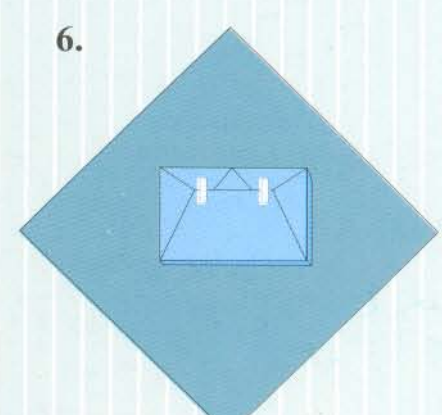

Fold back point and seal with adhesive tape

For further information feel free to contact us :

SURGEINE HEALTHCARE (INDIA) PVT. LTD.

101, Vardhman Tru Plaza, CU Block, Pitampura, New Delhi-110088 (India)

Tel. : +91-11-47552462, 27348301 | Telefax : +91-11-47552424

Mob. : +91-9350990935, 91-9310990931

E-mail : sales@surgeine.com

Website : www.surgeine.com

- CE 1023
- ISO 11607
- EN 868-2
- ISO 13485
- GMP
- ISO 9001